

INSTRUCTIONS
QYK-SET Acrylic Powders
(Denture Repair Kit, Intro Kit, Max Kit)

1. Assemble parts of broken denture with sticky wax.
2. Pour stone or plaster model.
3. Paint model with masque or tin foil medium.
4. Enlarge fracture line to 2-3mm and bevel an undercut to 45° angle.
5. Clean and set back on model.
6. Re – seated denture can now be repaired using this paint-on technique (powder and liquid in separate dappen dishes) or salt and pepper technique

IMPORTANT:

Due to heat generated during setup, do not allow material to set in mouth unless it is flushed with cold water to dissipate heat.

REPLACING BROKEN OR DISLODGED TEETH:

1. For single tooth replacement, bur out remaining portion of tooth from the lingual side. Select replacement tooth carefully so it fits accurately.
2. To replace two or more teeth, Bur out the remaining portions from the lingual side and cut away sufficient material so teeth can be easily set.
3. Select replacement teeth and with wax, set up teeth and fill in until original contour of denture is restored.
4. With plaster, IR compound, make it liable matrix.
5. When matrix is set, remove teeth and clean off all wax from teeth and repair area.
6. Using paint-on technique or dusting method, add QYK-SET® Acrylic Powder
7. Place in warm water (110°f) or use pressure.
8. Trim and polish when set (2 minutes!)